

PAW  WAVE™



Improve Mobility



Promote Bonding



Reduce Tension



Decrease Stress

You might have heard of our human products by PADO. We've taken our expertise in percussion & vibration massage and developed the first ever pet massager. Why? Because we love our Pets!

Just like you, we do everything we can to keep our pets healthy and happy. We've worked with everyone from engineers, to veterinarians, to the animals themselves all to bring the benefits of massage to our furry friends.

Want to meet our dream team of humans and happy pets?

Learn more about us at:
www.pawwave.pet

PAW WAVE PERK

THERAPEUTIC PERCUSSION PET MASSAGER



Using the PERK



For best results, charge the PERK before use.
(Approximately 180 mins for Full Charge)

Insert charging cable into the port on the side of your device.

Plug the charger into an outlet. The LED on the unit will turn on. The white LED lights will indicate charge level when unit is plugged into the charger. The LED indicator will turn off when the battery is fully charged.



Press and hold the power button to turn on the PERK.

Press the power button to adjust percussion mode speed. White LED lights on the massager will now indicate speed level.



Press and hold the power button to turn off the PERK.

Safety Feature: There is a 15 minute automatic shut-off for safety. If you need to use it longer, simply power it back on and massage away.



Calibrated for Pets

PERK brings deep tissue relief to your pet, with pet calibrated tips to address a wide range of needs. Good for helping to prevent injury, delay the onset of arthritis and hip dysplasia & improve arthritis stiffness. Also good for helping with post surgery rehab.

How to Attach

Line up attachment with the protrusion on the percussion end of the massager.

Screw on clockwise until secure.

Unscrew counterclockwise to remove.



3D Flex Tip

Flexing and rebounding of the arms recoils to amplify the effects of massage nodes to provide a powerful but forgiving impact. The flexible arms wrap around your pet's narrow legs and boney joints.

Soft Brush Tip


The soft silicone brushes are exponentially amplified by the effects of the percussive motion to the beat of 110 hz frequency. In other words, your pet will love this extreme massage-scratch session.



Cupping Tip

Designed to trap air in the diaphragm of the cup and deliver a deep percussive energy to the inner muscle belly while minimizing the impact blow to the skin.

WHAT'S IN THE BOX?

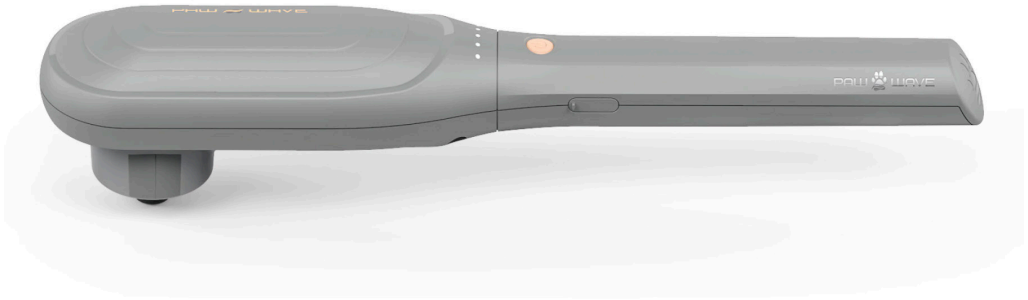

PERK Massager



3D Flex Tip



Cup Tip



Soft Brush Tip



Charger & Cable



Pouch

For FAQs and other tips for using your
PAW WAVE PERK visit:

www.pawwave.pet

PADO INC.

28340 Ave Crocker, Suite #100
Santa Clarita, CA 91355